

CLEANING INSTRUCTIONS

Starsys™ Electronic Touch Pads, cart bodies, and some accessories are constructed of durable advanced composite of polymers. A soft damp cloth with a mild soap solution can be used to clean soiled surfaces on the touch pad. Dry immediately with a soft dry cloth. Be careful not to get excess liquid on the touch pad itself. Always insure that the cleaner is recommended for use on plastics, and follow any special instructions from the manufacturer.

The use of abrasive cleaners is NOT recommended.

Please read the user's manual for recommendations on correct cleaning materials and procedures.

Note 1: The display will shut off if a user does not hit a key for 15 seconds.

Note 2: The key lock should be in the locked position; i.e., the key is vertical and aligned with orange dot to utilize the touch pad.

Note 3: The default auto re-lock time is 60 seconds. It can be set from 10 seconds to 999 minutes.



DEFAULT CODES:

Supervisor Code — 1350

User Code — 0531

Up to 2,000 available user ID's can be programmed.

Auto Re-lock default is 60 seconds (can be programmed from 10 seconds to 999 minutes)



STARSYS™

FULL FEATURE TOUCH PAD OPERATION GUIDE

Starsys™ Full Feature Touch Pad offers an advanced security system for carts that includes keyless unlocking, automatic re-locking, and a USB port to download/upload cart data and User codes.



BASIC TOUCH PAD OPERATION

TO UNLOCK UNIT USING TOUCH PAD

- Press **OK** to wake up touch pad
- Enter 4-digit User Code & Press **OK**
- If the code is valid, system will automatically unlock unit

TO UNLOCK UNIT USING CARD READER

- Press **OK** to wake up touch pad
- Scan/Swipe Card through Reader (proximity, magnetic stripe)
- If the card is valid, cart will automatically unlock

TO LOCK UNIT

- Press **SIGN OUT** on the touch pad

TO ADD USER CODE (Supervisor Only)

- Press **OK** to wake up touch pad
- Enter Supervisor Code & Press **OK**
- Press **FUNCTION** key until "Assign a New Operator ID" is displayed
- Press **OK**
- Follow prompts
- Press **SIGN OUT** to exit Supervisor Mode

TO DELETE USER CODE (Supervisor Only)

- Press **OK** to wake up touch pad
- Enter Supervisor Code & Press **OK**
- Press **FUNCTION** key until "Delete an Operator ID" is displayed
- Press **OK**
- Follow prompts
- Press **SIGN OUT** to exit Supervisor Mode

TO ADD USER CARD (Supervisor Only)

- Press **OK** to wake up touch pad
- Enter Supervisor Code
- Press **FUNCTION** key until "Assign a New Operator ID" is displayed
- Press **OK**
- Follow prompts
- Press **SIGN OUT** to exit Supervisor Mode

TO DELETE USER CARD (Supervisor Only)

- Press **OK** to wake up touch pad
- Enter Supervisor Code
- Press **FUNCTION** key until "Delete an Operator ID" is displayed
- Press **OK**
- Follow prompts
- Press **SIGN OUT** to exit Supervisor Mode

TO ADD SUPERVISOR CODE (Set Up Code Only)

- Press **OK** to wake up touch pad
- Enter Set Up Code & Press **OK**
- Press **FUNCTION** key until "Assign a New Manager ID" is displayed
- Press **OK**
- Follow prompts
- Press **SIGN OUT** to exit

TO DELETE SUPERVISOR CODE (Set Up Code Only)

- Press **OK** to wake up touch pad
- Enter Set Up Code & Press **OK**
- Press **FUNCTION** key until "Delete a Manager ID" is displayed
- Press **OK**
- Follow prompts
- Press **SIGN OUT**; to exit Set Up Mode

TO CHANGE AUTO RE-LOCK TIMER (Supervisor Only)

- Press **OK** to wake up touch pad
- Enter Supervisor Code & Press **OK**
- Press the **FUNCTION** key until "Set the cart time out" displays
- Press **OK**
- Follow prompts
- Press **SIGN OUT** to exit

TO CHARGE BATTERY

- Plug the recharger into a standard 120V AC wall outlet.
- Plug the recharger jack into the 12" long breakaway extender, and the male end of the extender into the jack on the back of the cart. (Extender will help to prevent drive-away damage if the cart is moved while connected to the recharger.)
- While the battery is charging, the light on the recharger will be red.
- When the battery is fully charged, the recharger light will turn to green.
- Initial charge time can be up to 6-8 hours.
- To check the battery level, a user can press and hold the **SELECT** key on the touch pad. 12.0 Volts is the approximate level where you will start to see a **LOW BATTERY — RECHARGE** message on the touch pad.
- Recommended to recharge every 3 to 4 days. If possible charge daily (nightly). You can get approximately 1.5 years of use before changing batteries if the system is taken care of properly.