

User Guide

Lifeline Emergency Cart



How to use your Defibrillator Swing Arm



- To adjust tray turn knob on the front of the tray clockwise to tighten and counter-clockwise to loosen.
- Turn tension knob on the rear of the cart counter-clockwise, platform rotates freely.
- To secure arm in place during transit, turn knob on the rear post clockwise.
- To modify swing arm tension, adjust knob on the rear post.

How to Maneuver Your Lifeline Emergency Cart

STATIONARY HANDLE

Stationary handle with trigger provides 1-step activation. Squeeze trigger to activate 5th wheel, let go to release.



5TH WHEEL STEERING

After engaging the trigger, a 5th wheel rotates to a rigid position enabling easy turning and maneuverability.



Two brake casters located in front of cart. Provides convenient access to immobilize the cart, when needed.

SECURITY SEAL LOCKS

ONE-SEAL

Tamper evident seal, located on the top of the cart, secures the ENTIRE cart. Can also be used to seal the top bin only, in two-seal configuration.



TWO-SEALS

Tamper evident seals secure the top compartment independently from the rest of the cart. The second seal secures the remainder of the cart. (Top compartment seal can be broken without breaking bottom seal; depressing bottom latch breaks both security seals.)



Top Compartment:

The top compartment cover slides easily towards the rear and hangs freely on the support wire.



Locking Tilt-Out Bins

Optional locking tilt-out bins are secured when cart is in locked position. Open tilt-out bins may be closed after cart is locked.

Maintenance Guide

Lifeline Emergency Cart



Locating the Serial Number on Lifeline

The serial number is located on the back panel of the cart.

Cleaning Instructions

Lifeline carts are constructed of a durable polyethylene. A soft cloth and mild soap and water solution can be used to clean lightly soiled surfaces. After cleaning, wipe dry with a clean, soft cloth.

For heavily soiled areas, a soft brush and solvent or emulsion-based cleaners may be employed. Always insure that the cleaner is recommended for the use on plastics and follow special instructions from the manufacturer.

The use of abrasive cleaners is NOT recommended. Iodine-based cleaners are NOT recommended and may cause discoloration to the cart.

Please read the instructions that come with the cart for recommendations on correct cleaning materials and procedures.

Recommended Types of Cleaning Products:

Note: This list is by no means complete. The listed products are only suggested and have been proven effective in cleaning the cart body and drawers.

Backboard Inspection

Because of the critical nature of cardiac backboards and the environments in which they are used, proper performance is essential.

Metro recommends that backboards should be inspected after each use and replaced if chipped, scratched, or gouged. If a damaged board is used while performing cardiopulmonary resuscitation, it is possible that the board could break and not provide proper support.

The physical nature of usage and varying frequency of the use makes it difficult to determine the life expectancy of a board. As a baseline, we are recommending replacement of all boards 5 years of age or older. Due to the risk of failure as a result of damage caused by impact and/or other forms of abuse, regular inspection should be incorporated into biomedical maintenance and quality assurance protocol.

You can determine the age of your Metro Backboard by calling customer service with the serial number found on the back of the cart. Old models may have the serial number located under the white movable handle on the right side of the cart.

CAUTION: DO NOT clean tilt-out bins with Quaternary Ammonia Compounds!

Product	Description	Base
Amphyl	Liquid detergent, disinfectant, deodorizer	Phenol
Bleach	Liquid disinfectant	Chlorine
Conquer II	Liquid detergent	Alkaline
FSD-4	Liquid detergent, disinfectant	Quaternary, Ammonia Compound
Mirco-Quat	Liquid detergent, germicide, deodorizer	Quaternary, Ammonia Compound
Prime	Liquid detergent	Alkaline
Spartec	Liquid detergent, disinfectant	Quaternary, Ammonia Compound
Spra-Temp	Foam cleaner and disinfectant	Quaternary, Ammonia Compound